

## **JOBST foam™**

The comfort of compression,  
anywhere you need it



**Foam rubber pad  
and bandage**



# JOBST foam™

The comfort of compression,  
anywhere you need it.

## Application Methods



Hold pad firmly in place while wrapping with appropriate short stretch compression bandage (Comprilan®) or,

Pull compression stocking onto foot. Pull heel pocket over back of heel. Tuck pad under fabric with flat side away from skin. Ease stocking over pad.



Without tension, apply the foam over an elastic tubular bandage (Tricofix®) or synthetic padding (Artiflex®).



Cut the foam bandage or use the foam pad to fit required shape. Cover the foam bandage with a short stretch compression bandage (Comprilan®) and wrap as necessary.

## Combine JOBST foam with any of these JOBST products to successfully manage Lymphoedema.



JOBST products



100% cotton short stretch bandage

## Overview

**Jobst foam** is designed to evenly distribute the pressure from compression stockings or bandages over bony prominences. Without any natural rubber latex, **Jobst foam** is available in two versatile forms – a foam rubber bandage and a foam rubber kidney-shaped pad.



## Use

**Jobst foam** is versatile and comfortable enough to fit any contour of the body, especially where consistent compression is required.

- Apply the foam rubber pad to maintain compression around the ankle, the hand or other sensitive areas.
- Apply the foam rubber bandage as a wrap to manage edema, as additional padding under compression bandages or as a layer of protection for the skin.
- The foam rubber roll can be used as one large bandage or cut and customized into specific shapes and/or sizes.
- Use with caution in cases of
  - Skin infections
  - Wet dermatoses
  - Impaired sensitivity of the legs
- Improper application may result in problems when wearing the bandage



## Specifications

Product	Ref. Number	Description	Packaging units
JOBST foam rubber kidney-shaped pad	78496-00	9 cm x 1 cm	1 Pad / Box 75 Boxes / Case
JOBST foam rubber bandage	78497-00	10 cm x 1 m x 1 cm	1 Bandage / Box 20 Boxes / Case